# Shapes<sup>®</sup> by PolyMem<sup>®</sup>

QuadraFoam<sup>®</sup>

REF 8015

## #1 Oval

REF 8023

## #3 Oval

### Directions for Use

**AU REP**

Emergo Asia Pacific Pty Ltd  
201 Sussex Street  
Darling Park, Tower II, Level 20  
Sydney, NSW 2000 Australia

**EC REP**

Emergo Europe  
Molenstraat 15  
2513 BH The Hague  
The Netherlands

**FERRIS MFG. CORP.**

16W300 83rd Street  
Burr Ridge, IL 60527-5848 U.S.A.  
630-887-9797

Toll Free U.S.A.: 800-765-9636  
International: +1 630-887-9797

e-mail: [info@ferrispolymem.com](mailto:info@ferrispolymem.com)

Website: [www.PolyMemShapes.com](http://www.PolyMemShapes.com)

Website: [www.PolyMemShapes.eu](http://www.PolyMemShapes.eu)

Covered by one or more U.S. and similar international patents:  
4,884,563; 5,064,653; 5,065,752; 5,254,301; 5,916,928; 5,605,165;  
6,043,406; 6,346,653; 6,349,828; 6,350,339; 7,078,055; 7,078,056.  
Other patents pending

## PRODUCT DESCRIPTION

**Shapes™ by PolyMem®**, a **QuadraFoam™** Wound Dressing, is composed of a patented hydrophilic polyurethane membrane matrix with a thin film adhesive border. The membrane pad is covered with a semipermeable continuous thin film backing which is optimized for oxygen and moisture vapor permeability and as a barrier to liquids. The PolyMem formulation is the **only** dressing formulation which contains (1) a safe, non-toxic cleanser (F68 surfactant), (2) a moisturizer (glycerol, also known as glycerin), and (3) an absorbing agent (superabsorbent starch copolymer), all in the pad matrix. Both F68 and glycerol are soluble in wound fluid and skin moisture. The superabsorbent starch copolymer contained in the PolyMem formulation draws wound fluid, which is known to contain natural growth factors and nutrients, to the wound site.

### All PolyMem Formulation Dressings Relieve Pain

#### INDICATIONS

For the management of:

- Pressure ulcers (Stages I-IV)
- Acute wounds
- Donor and graft sites
- Diabetic ulcers
- 1st and 2nd degree burns
- Venous stasis ulcers
- Leg ulcers
- Skin tears
- Dermatologic disorders
- Surgical wounds

PolyMem formulation dressings are suitable for use when visible signs of infection are present if proper medical treatment that addresses the cause of the infection has been implemented.

#### CONTRAINDICATIONS

Do not use on patients with known sensitivity to any ingredient.

PolyMem formulation dressings are not compatible with hypochlorite solutions (bleach) such as Dakin's solution.

#### PRECAUTIONS

- For external use only

## DIRECTIONS FOR USE

### APPLICATION

- 1. INITIAL DRESSING APPLICATION:** Prepare the wound according to protocol or as directed by a physician or other ordering clinician.
- Select a dressing so that the membrane is approximately 1/4" – 2" (0.6 – 5 cm) larger than the wound.

**HELPFUL HINT: For dry wounds, moisten dressing slightly with sterile water or saline prior to application.**

3. Apply the dressing film side out and/or printed side out.
- 4. DRAMATIC** increase in wound fluid may be observed during the first few days. This is not uncommon, and indicates that the dressing is working.

**HELPFUL HINT: Outlining the wound size on the outside of the dressing helps in determining when a dressing change is needed.**

5. Keep the dressing **DRY** and in place when bathing. Change the dressing if it gets wet.
6. Topical treatments are not recommended in conjunction with Shapes dressings.

### DRESSING CHANGE

1. Change dressings before the wound fluid, visible through the dressing, reaches the wound margin. Change the dressing immediately if the wound fluid reaches the edge of the membrane pad. For mildly exudating wounds in otherwise healthy patients, dressing can be left on for a recommended maximum of seven days. As with other dressings, more frequent changes may be indicated if the patient has a compromised immune system, diabetes, or infection at the wound site.
2. Gently peel the adhesive film toward the center of the dressing.
3. Remove the dressing from the wound site. The PolyMem formulated pad will not adhere to the wound bed, usually assuring pain-free dressing changes.
- 4. Do not disturb the wound bed. Do not clean the wound bed** or flush with saline or water unless the wound is infected or contaminated. PolyMem formulation dressings contain a wound cleanser and leave no residue. Additional cleaning of the wound may injure regenerating tissue and delay wound healing. The intact skin around the wound may be cleansed while leaving the wound bed undisturbed.
5. Apply a new dressing.

**STORAGE:** Keep dry and store at room temperature and humidity.

**DISPOSAL:** Dispose of used dressings as typical biohazardous plastic material.

**Shapes™ by PolyMem® #1 & #3 Ovals were created to provide water resistant dressings shaped to accommodate smaller wounds.**


## **PREPARATION**

The periwound should be clean and dry. Use of a protective barrier wipe will help the dressing adhere to the skin. Moisturizers, lotions, ointments or creams should not be used. They will interfere with the film adhering to the skin.


## **APPLICATION**

The dressings are easily applied. Both the #1 and the #3 Oval dressings share the same application method.


1. Turn the dressing over so that the release liner is facing up. Bend the dressing along the score line, marked by a red stripe, to crack and separate the liner.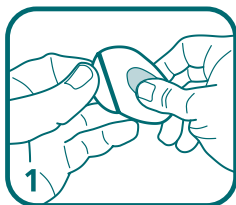


2. Peel away the large portion of the liner while holding the dressing by the remaining small liner.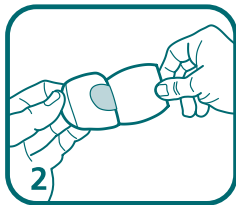


3. Apply the dressing being careful to avoid wrinkling of the film if possible.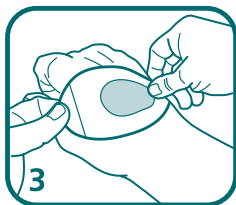

4. Remove the remaining portion of the liner.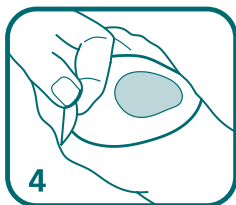

5. Smooth the dressing onto the skin to assure a good bond.

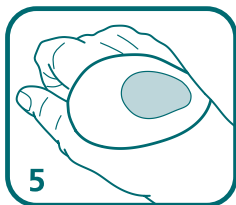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